



Xpert™ MRSA Specimen Collection Protocol


Xpert™ MRSA

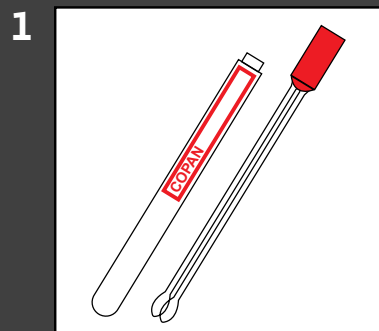


A Copan Venturi Transystem double-swab must be used to collect the specimen.
Cepheid Collection Device #900-0370

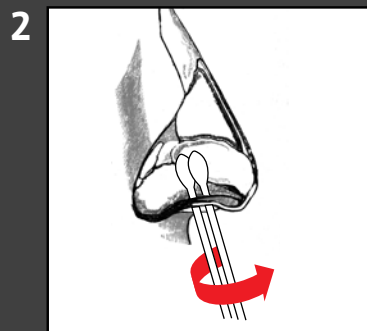


Insert the dry swabs 1–2 cm into the nostril and rotate swabs against the inside of the nostril for 3 seconds while applying pressure with a finger to the outside of the nose.

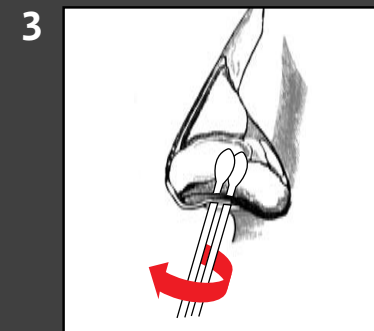

Repeat Step 2 in second nostril with the same swabs.

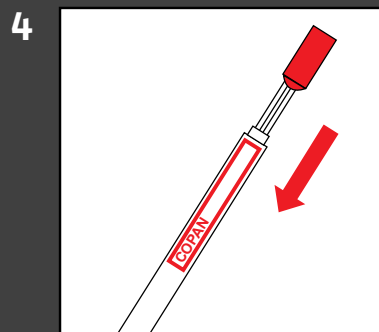

Place the swabs back into the tube.

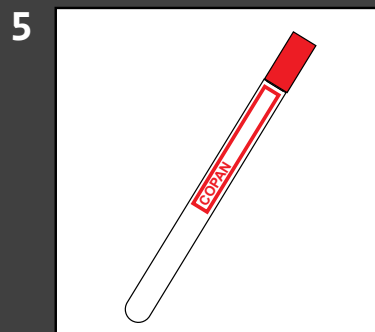

Specimens that can be tested within 24 hours can be kept at room temperature. If not, it is recommended that they be refrigerated. Specimens stored at 2–8° C are stable for up to 5 days.

defining *on-demand* molecular diagnostics.

 **Cepheid**[®]
Bring answers to life.

For Technical Assistance please contact: Cepheid Technical Support: 888-838-3222, Option 2